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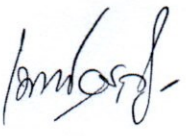
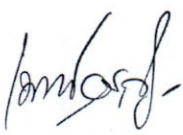
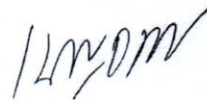
15-20 % hem

myoglobin, cytochromes, catalases

metabolism bilirubin unconjugated

conjugated bilirubin hyperbilirubinemia

hyperbilirubinemia

- Prehepatic jaundice : unconjugated

hyperbilirubinemia corpuscular hemolytic anemia thalassemia

sickle cell anemia, extra corpuscular hemolytic anemia blood transfusion reaction

ABO/Rh incompatible, Neonatal jaundice and hemolytic disease of the

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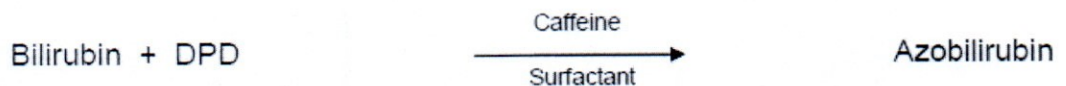
- **Hepatic jaundice :** conjugated hyperbilirubinemia  
acute and chronic viral hepatitis, liver cirrhosis, hepatocellular carcinoma
- **Post hepatic jaundice:** conjugated hyperbilirubinemia  
extrahepatic cholestasis, liver transplant rejection

Chronic congenital hyperbilirubinemia  
unconjugated hyperbilirubinemia  
Crigler-Najjar syndrome, Gilbert's syndrome,  
conjugated hyperbilirubinemia  
Dubin-Johnson syndrome, Rotor syndrome  
chronic congenital hyperbilirubinemia  
acquired type  
bilirubin fraction  
acquired

2. Principle of the procedure used for examination)

3,5-Dichlorophenyl diazonium tetrafluoroborate (DPD) conjugate  
bilirubin  
unconjugate bilirubin from azobilirubin  
total bilirubin  
540nm  
dark

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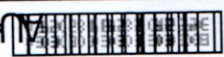
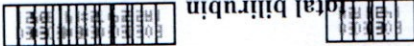
 90

 Total bilirubin

Coffee 21mg/L  
 3,5-Dihydroxyindoleacetic acid (DHI) 0.31mg/L  
 Surfactant  
 Preservative

63  Centrifuge

64 

641 Standard Uses System Calibrator Cat. No. 6630

642  Calibrator

System Calibrator Cat. No. 6630

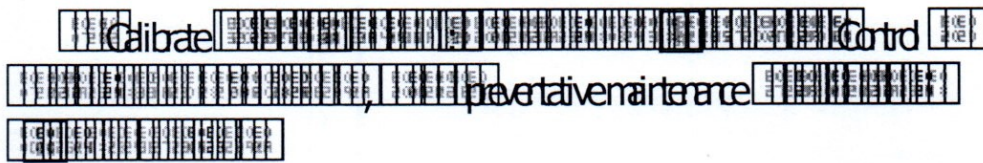
 Total bilirubin

Standard Technology (NIST) Standard Reference Material (SRM) 916a

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	-20°C *	2...8°C
Alkaline Phosphatase, ALT, AST, Amylase, Cholinesterase, CK-NAC, GGT, HBDH, LDH	Not recommended	8 hours
Albumin, Creatinine, Cholesterol, Glucose, Lactate, Lipase, Total Protein, Triglyceride, UIBC, Urea, Uric Acid, ** Bilirubin, *** Inorganic Phosphorous	1 month	30 hours
Calcium, Iron, Magnesium	1 month	1 week

\* When frozen once.

7. (Calibration)
- 7.1 System Calibrator Cat. No 66300
- 7.2 Calibrator 14 Control
- (Sift)
- 

8. (Procedural steps)
- AU180/AU680/AU5800
- AU180/AU680/AU5800

9. (Quality control procedures)
- 9.1 000003 000004
- 9.2
- Calibration

10. (Interferences)
- Hemolysis Interference less than 10% upto 0.45 g/L haemoglobin

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### Lipemia Interference less than 10% upto 1000mg/dL. Irtalipid®

11.  (Principle of procedure for calculating results, including measurement uncertainty)

12.  (Biological reference intervals)

Serum (Adults) 5–21  $\mu\text{mol/L}$  (0.3–1.2  $\text{mg/dL}$ )

Serum (Children)

0–1 day 24–149  $\mu\text{mol/L}$  (1.4–8.7  $\text{mg/dL}$ )

1–2 days 58–197  $\mu\text{mol/L}$  (3.4–11.5  $\text{mg/dL}$ )

3–5 days 26–205  $\mu\text{mol/L}$  (1.5–12.0  $\text{mg/dL}$ )